Bijsluiter

Information for use

Nederlands

English

Putty Buddies

Ear Band-It

Ear Band-It ULTRA

2025/2026

voor
je oor

PUTTY
BUDDIES®

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How to use

Putty Buddies soft silicone ear plugs were invented by an Ear, Nose, Throat physician with over 40 years experience and are recommended by doctors worldwide for swimming, showering, and general water protection.

WARNINGS

Adult supervision required. Not recommended for diving depths greater than 10 feet. Not recommended for use with swim caps. Not recommended for noise prevention or sleeping. Avoid contact with hair, lotions, sunscreens, or chemicals. Handle with clean and dry hands. Store plugs in controlled room temperature environment while not in use. For maximum protection, ear plugs should be worn with the Ear Band-It or Ear Band-It ULTRA swimming headband.

CAUTION

Only cover opening of ear. DO NOT INSERT INTO EAR CANAL. Silicone ear plugs are safe when used as instructed. If ear plugs are pushed into ear canal, DO NOT try to remove yourself. Do not use tweezers, cotton swabs, or other devices as these may push the plug further into the ear canal. See an Ear, Nose, Throat (ENT) Physician as they have proper instruments and skill to remove the plugs safely. A silicone ear plug in the ear canal for a short time may be harmless, providing you do not attempt to remove the ear plug yourself and accidentally push it deeper into opening. Use only as directed.

IMPORTANT!

To be effective, the Ear Band-It must be properly worn with ear plugs or ear molds. Not recommended for diving head first.

WARNING CHOKING HAZARD

Small parts. Use only under adult supervision. Store away from children under 3 years old when not in use.

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How to use

- Plugs should be applied by an adult, never a child.
- Shape whole plug into ball. Flatten with fingers. Shape to cover outer ear until snugly fit.
- Loosen and remove plugs by pressing up behind the ear.
- Plugs are reusable. Discard 2 weeks after initial use, (max 5 uses) or if plugs crack, separate or become dirty.
- **DO NOT** tear plugs apart or force into ear canal.
- **DO NOT** use a swim cap.
- **DO NOT** use for sleep or noise prevention
- **DO NOT** make contact with hair, lotions, sunscreens, or chemicals. Handle with clean and dry hands.

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How to use

Directions

Headband crosses at or below the hairline. Secure behind the neck for best results. Note that the headband may be skightly tight when first putting it on. If so, grab each end and pull firmly several times to stretch out the neoprene.

Care instructions

The Ear Band-It will stay clean in chlorinated pools, bathtubs, or showers. If washing is needed, hand was with mild detergent and air dry.

IMPORTANT!

To be effective, the Ear Band-It must be properly worn with earplugs or ear molds. Not reccomended for diving head-first.

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Why Ear Band-It?

The Ear Band-It was invented by an Ear, Nose, Throat physician with over 40 years' experience and is recommended by doctors worldwide for swimming, showering, and general protection. It was created to ensure ear plugs or ear molds stay in place and keep water out of the ears. The unique design of the Ear Band-It offers ultimate comfort and protection for the ears.

Initially the Ear Band-It was marketed to children with ear tubes, enabling them to swim. This has since been expanded to include water enthusiasts of all ages who have swimmer's ear, ear drum perforations, or simply do not like to get water in the ears.

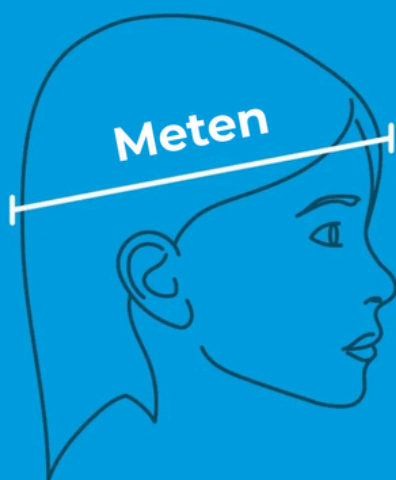
Studies has shown that swimmer's ear (otitis externa) affects 1 in every 125 people, while nearly one million people have tubes placed in their ears annually in the USA alone. Prior to the Ear Band-It, the only option for these affected individuals was the bathing cap, which was not only unsightly and uncomfortable, but often times ineffective in protecting the ears.

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
# How to measure for the right Ear Band-It size
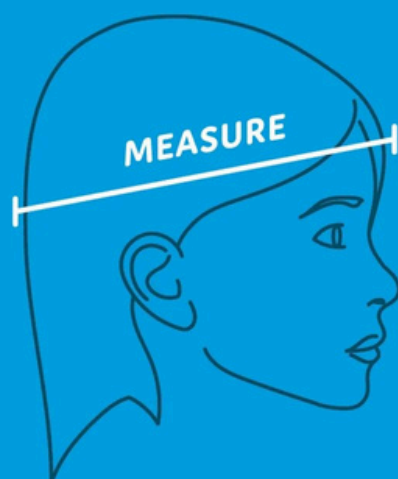
## MEASURE HEAD CIRCUMFERENCE PRIOR TO ORDERING!

Sizing issues and the hassle of returning items can be avoided simply by measuring head circumference and referring to the size chart. Using a soft tape measure is best but a string and standard tape measure will work as well.

|               | Fits Head Size Range               | Recommended Age Range* |
|---------------|------------------------------------|------------------------|
| <i>Small</i>  | 16.5 in – 18.5 in<br>42 cm – 47 cm | Up to 18 months        |
| <i>Medium</i> | 18.5 in – 20.5 in<br>47 cm – 52 cm | 2 – 7 years            |
| <i>Large</i>  | 20.5 in – 24 in<br>52 cm – 61 cm   | 8 years to adult       |

~~~~ NOTE THAT RECOMMENDED AGE RANGES ARE BASED ON AVERAGE SIZED HEADS. ~~~~

*We often see kids with larger than average head sizes needing to go up one size if only referring to age recommendations. **AGAIN, IT IS ALWAYS BEST TO MEASURE AND CHECK THE SIZE CHART!**

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How to place the Ear Band-It

1. Preparing the ears

If you are using Putty Buddies swimming earplugs, flatten them and gently place them over your ear canal before applying the Ear Band-It.

2. Applying the Ear Band-It

Wrap the Ear Band-It around your head, ensuring both ears are completely covered. The Ear Band-It will cross comfortably at or just below your hairline.

3. Adjusting the fit

Attach the Ear Band-It to the back of your neck for an optimal and comfortable fit. Use the Velcro to keep the band securely in place.

4. Checking the comfort

Check that the Ear Band-It is not too tight, but not loose enough to slip. For best results, wear the Ear Band-It under your hair.

This will ensure the Ear Band-It stays in place while swimming.

5. Removal and storage

Carefully remove the Ear Band-It after use.

Store the Ear Band-It in a dry, clean place to extend its lifespan.

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Materials

The Ear Band-It is made of neoprene and is non-toxic, BPA-free, and latex-free.