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Swimming headband

In combination with soft and/or moldable earplugs, such as Putty Buddies, the Ear Band-It holds earplugs firmly in place and effectively prevents water from entering the ears.

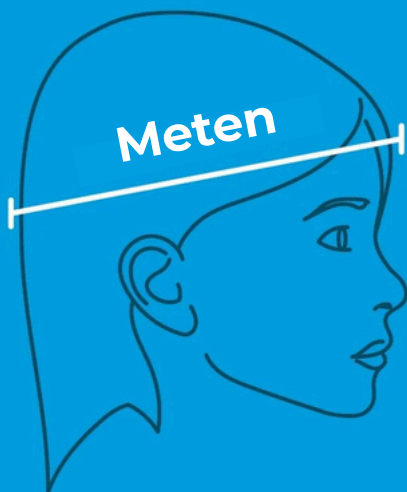
The swimming headband helps prevent ear infections as well as swimmer's ear! Ideal for children with ear tubes, a perforated eardrum, and other ear conditions. Can also be worn with a cochlear implant provided it is used in combination with the implant's swimming protection.

Only effective in combination with soft and/or moldable earplugs

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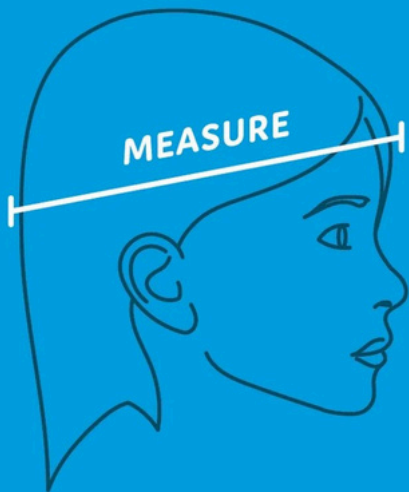
ADJUSTABLE
NEOPRENE
SWIMMING
HEADBAND

Measure the head circumference before ordering!

Problems with sizing and the hassle of returning items can easily be avoided by using head circumference, as indicated in the size chart. A soft measuring tape works best, but a piece of string with a standard measuring tape works just as well.

	Fits Head Size Range	Recommended Age Range*
<i>Small</i>	16.5 in – 18.5 in 42 cm – 47 cm	Up to 18 months
<i>Medium</i>	18.5 in – 20.5 in 47 cm – 52 cm	2 – 7 years
<i>Large</i>	20.5 in – 24 in 52 cm – 61 cm	8 years to adult

NOTE THAT RECOMMENDED AGE RANGES ARE BASED ON AVERAGE SIZED HEADS.

*We often see kids with larger than average head sizes needing to go up one size if only referring to age recommendations. **AGAIN, IT IS ALWAYS BEST TO MEASURE AND CHECK THE SIZE CHART!**

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ADJUSTABLE
NEOPRENE
SWIMMING
HEADBAND

How to use

Directions: The headband crosses the hairline at or below the hairline. Secure the headband at the back of the neck for a comfortable, snug fit. Wear the headband under the hair for best results. Please note that the headband may feel slightly tight at first. In that case, grasp both ends and pull firmly a few times to stretch the neoprene.

Care Guidelines: The Ear Band-It stays clean in chlorinated water in swimming pools, bathtubs, or showers. If washing is necessary, hand wash with a mild detergent and air dry.

Material: The Ear Band-It is made of high-quality neoprene and features an adjustable Velcro closure for a comfortable and secure fit.

Important! To be effective, the Ear Band-It must be worn correctly with moldable and/or soft earplugs. Not recommended for headfirst diving.

Why Ear Band-It?

The **Ear Band-It** was invented by an Ear, Nose, Throat physician with over 40 years' experience and is recommended by doctors worldwide for swimming, showering, and general water protection. It is designed to hold moldable and soft earplugs securely in place and keep water out of the ears. The unique design of the **Ear Band-It** offers comfort and protection for the ears.

Originally, the **Ear Band-It** was designed for children with ear tubes, enabling them to swim. This has since expanded and now includes water enthusiasts of all ages with swimmer's ear, perforated eardrums, or simply do not like to get water in the ears. Can also be worn with a cochlear implant.

Studies have shown that swimmer's ear (otitis externa) affects nearly 1 in 125 people, while nearly a million people in the US alone receive ear tubes annually. Before the introduction of the **Ear Band-It**, the only option for these affected individuals was the bathing cap, which was not only unsightly and uncomfortable, but often times ineffective in protecting the ears.

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How to put on the Ear Band-It

1. Preparing the ears

Always use soft and/or moldable earplugs in combination with the Ear Band-It, such as Putty Buddies. Apply the Putty Buddies as described in the IFU before putting on the Ear Band-It.

2. Applying the Ear Band-It

Wrap the Ear Band-It around the head, ensuring that both ears are fully covered. The Ear Band-It crosses comfortably at or just below the hairline.

3. Adjusting the fit

Secure the Ear Band-It at the back of the neck for an optimal and comfortable fit. Use the Velcro strap to keep the band firmly in place.

4. Checking comfort

Check that the Ear Band-It is not too tight, but also not loose enough to shift. For best results, wear the Ear Band-It under your hair. This ensures the Ear Band-It stays perfectly in place while swimming.

5. Removal and storage

Carefully remove the Ear Band-It after use. Store the Ear Band-It in a dry and clean place to extend its lifespan.

Please note: the Ear Band-It is only effective in combination with moldable and/or soft earplugs. The use of hard or custom-made earplugs is not recommended, as the Ear Band-It may push them out of the ear.

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© EarVolution Inc. 1992-2020