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# Swimming ear plugs

Meet the most comfortable extra sticky earplugs for use in the water that stay in place in the ears.

These soft, moldable silicone earplugs have been specially developed for optimal protection against water, ideal for children with ear tubes or to prevent ear infections.

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**3**

## How to use

- Plugs should be applied by an adult, never a child.
- Shape whole plug into ball. Flatten with fingers. Shape to cover outer ear until snugly fit.
- Loosen and remove plugs by pressing up behind the ear.
- Plugs are reusable. Discard 2 weeks after initial use, (max 5 uses) or if plug cracks, separate or become dirty.
- **DO NOT** tear plugs apart or force into ear canal.
- **DO NOT** use with swim cap.
- **DO NOT** use for sleep or noise prevention.
- **DO NOT** make contact with hair, lotions, sunscreens, or chemicals. Handle with clean and dry hands.

## How to apply Putty Buddies

- Flatten/mold plug over ear opening.
- **Do not** insert into ear canal.
- **Do not** tear plugs apart.
- Water use only.
- Adult supervision required.
- Visit ENT Physician if plug becomes stuck in ear canal.
- Keep away from infants.
- Non toxic but may pose choking risk.
- Non-allergenic.

1  
Open Putty Buddies container & remove earplug.

2  
Roll into a ball and slightly flatten between fingers and thumb.

Cover ear canal with Putty Buddies and press outward on the edge of the earplug to lock it into the cartilage of the outer ear canal and create a watertight seal - do not insert plug into the ear canal.

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## Why Putty Buddies

Putty Buddies soft silicone earplugs were invented by an Ear, Nose, Throat Physician with over 40 years' experience and are recommended by doctors worldwide for swimming, showering and general water protection.

**WARNINGS:** Adult supervision required. Not recommended for diving depths greater than 10 feet. Not recommended for use with swim caps. Not recommended for noise prevention or sleeping. Avoid contact with hair, lotions, sunscreens, or chemicals. Handle with clean, dry hands. Ear plugs should ONLY be worn while in water. Store plugs in controlled room temperature environment while not in use.\* For maximum protection, ear plugs should be worn with the Ear Band-It or Ear Band-It ULTRA swimming headband.

**CAUTION:** Only cover opening of ear. DO NOT INSERT INTO EAR CANAL. Silicone ear plugs are safe when used as instructed. If ear plugs are pushed into ear canal, DO NOT try to remove yourself. Do not use tweezers, cotton swabs, or other devices as these may push the plug further into canal. See an Ear, Nose, Throat (ENT) Physician as they have the proper instruments and skill to remove the plugs safely. A silicone ear plug in the ear canal for a short time may be harmless, providing you do not attempt to remove the ear plug yourself and accidentally push it deeper into the opening. Use only as directed.

**WARNING: CHOKING HAZARD** - Small parts. Use only under adult supervision. Store away from children under 3 years old when not in use.

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© EarVolution Inc. 1992-2020